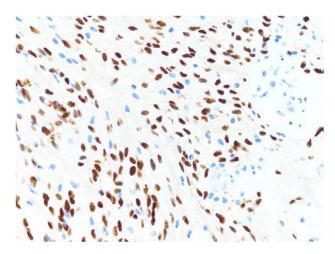


# Rabbit monoclonal antibody against **SOX10 (Clone ISR026)**



**Figure 1** Schwannoma stained with anti-SOX10 antibody.

#### **Product identification**

ISC124-R3 3 ml ready-to-use (RTU) ISC124-R7 7 ml ready-to-use (RTU) ISC124-C1 1 ml concentrated

# Summary and explanation

Transcription factor SOX10 belongs to the SOX [sex determination region Y (SRY) related high mobility group (HMG) box] gene family, which regulates Wnt/beta catenin signal in various development processes. The SOX10 gene is located on chromosome 22q13.1. It mediates the differentiation of neural crest cells into melanocytes, oligodendrocytes, and glial cells, promotes stem cell/progenitor cell activity, and induces mesenchymal transformation of mammary cells. SOX10 is expressed in many different cells and tissues, including melanocytes, Schwann cells. corresponding tumors. It is also found in some solid tumors - breast cancer, salivary tumor, hepatocellular carcinoma, ovarian tumor, nasopharyngeal carcinoma, prostate cancer and digestive system cancer. In addition, the expression of SOX10 in myoepithelial cells of the breast, salivary glands and bronchial glands has been confirmed through immunohistochemistry.

# Principle of the procedure

The stated primary antibody is suitable for immunohistochemical staining of FFPE tissue sections based on specific antigen-antibody reaction. Using a detection system linked to HRP or alkaline phosphatase the antigen visualization is performed via specific binding of the primary antibody. Secondary antibody is binding to the primary antibody, and the enzyme complex labels this complex. The enzymatic activation of the chromogen results in a visible reaction product at

the antigen site. Each step is incubated for a precise time and temperature and requires interposed washing steps. The specimen may then be counterstained. Results are interpreted using a light microscope.

Materials provided	
Primary antibody	SOX10
Host	Rabbit
Subclass	lgG
Clone	ISR026
Species Cross-reactivity	Human. Others-not
	known
Applications	Immunohistochemistry
Epitope Retrieval	Heat-induced epitope
	retrieval
Ready-to-use antibody	Prediluted antibody in
•	antibody diluent buffer
Recommended working	1:50 to 1:100
dilution range	

Product label shows the specific lot number.

Prediluted antibody is ready-to-use and optimized for staining. No further dilution, reconstitution, mixing, or titration is needed.

Antibody concentrate is optimized for dilution within dilution range using InSituChem® Antibody Diluent for IHC (Cat. No. D005-50). Indicated dilution range should be considered as recommendation and depends on different facts (tissue, fixation, incubation conditions, etc.). Optimum dilution to be determined in user's own system.

#### Materials required but not provided

The following materials may be required for staining but are not provided with the primary antibody.

- Positive and negative controls
- Microscope slides (positively charged) and cover slips
- Water bath
- Humidified chamber
- Staining jars
- Stopwatch
- Xylene or xylene substitute
- Ethanol
- Deionized or distilled water
- Antigen retrieval reagent, e.g. Antigen Enhancer (HIER buffer), Cat. No. D004-500
- Detection system, e.g. IHC Complete Detection system (Goat anti mouse/rabbit HRP, DAB staining), Cat. No. D001-15
- Wash buffer: e.g. IHC Wash Buffer, Cat. No. D003-500
- Tap water/bluing reagent (e.g. ammonia water)
- Light microscope

#### Storage and handling

Store at 2 - 8 °C.

SOX10 En\_Rev. 1 IM Page 1 of 2



When stored correctly antibody is stable to the expiration date indicated on the vial. Do not use after expiration date.

To ensure proper reagent delivery and stability of the antibody, replace the dispenser cap after every use and immediately place the bottle cool in an upright position.

# Staining procedure

- 1. Cut 3-4 µm section of formalin-fixed paraffinembedded tissue and place on positively charged slides.
- 2. Dry at 65°C for 2 hours.
- 3. Deparaffinize, rehydrate, and epitope retrieve. Upon completion, rinse with 3 changes of distilled or deionized water.
- 4. If using HRP detection system, place slides in peroxide block for 10 minutes; rinse. If using AP detection system, omit this step.
- 5. Apply the antibody and incubate for 60 minutes; rinse.
- 6. Apply the InSituVison™ Polymer Rabbit/Mouse Detection System for 30 minutes; rinse.
- 7. Apply ample amount of DAB or AEC chromogen and incubate; rinse.
- 8. Dehydrate and coverslip.

### Interpretation of results

The immunostaining procedure causes a colored reaction product to precipitate at the antigen sites localized by the primary antibody.

# Cellular localization: Cell nucleus.

qualified pathologist experienced immunohistochemistry procedures must evaluate positive and negative tissue controls before interpreting patient specimens.

Positive staining intensity should be assessed within the context of any background staining of the negative reagent control.

Note: A negative result means that the antigen in question was not detected, but not that the antigen is not present in the cells/tissues tested. An antibody panel may be used to support the results in some circumstances. Additionally, the morphology of each tissue sample should be examined utilizing a hematoxylin and eosin stained section. A qualified pathologist must interpret the patient's morphologic findings and pertinent clinical data.

# Warnings and precautions

- 1. Application only by qualified and trained personnel.
- 2. There are no estimated health risks, if the product is used as directed. MSDS is available on request.
- 3. Product contains sodium azide as preservative. Pure sodium azide is toxic. The concentration of sodium azide in this reagent is < 0.1 % and is not classified hazardous. See MSDS.

- 4. As with any product derived from biological sources, proper handling procedures should be used.
- Do not use reagents after expiration date.
- Take reasonable precautions when handling reagents. Use protective clothing and gloves.
- 7. All hazardous materials should be disposed according to guidelines for hazardous waste disposal. Materials of human or animal origin should be handled as biohazardous materials and disposed of with proper precautions.
- 8. Avoid microbial contamination of reagents as it may cause incorrect results.

# Literature

- [1] Gubbay J, Collignon J, Koopman P, et al. A gene mapping to the sex-determining region of the mouse Y chromosome is a member of a novel family of embryonically expressed genes[J]. Nature 1990, 346:245-50.
- [2] Haldin CE, LaBonne C. SoxE factors as multifunctional neural crest regulatory factors[J]. Int J Biochem Cell Biol 2010, 42:441-4.
- [3] Dravis C, Spike BT, Harrell JC, et al. Sox10 regulates stem/progenitor and mesenchymal cell states in mammary epithelial cells[J]. Cell Rep 2015, 12: 2035-48.

#### Date of publication or revision

2024-06-10 Change(s) made: -

# **Explanation of symbols**



Catalog number



Batch code



Use by



Temperature limitation



Do not use if package damaged



Consult instructions for use



Caution

SOX10 En Rev. 1 IM Page 2 of 2